

2020-9-16 批号: 2020091601  
 KOUMASK\_ (KKF-1A-I) 非医用(塑料袋贴纸)  
 EN 149:2001+A1:2009 CE0370北测  
 内容印刷尺寸: 100\*150mm

C100 M50 Y0 K0

		Model <b>KKF-1A-I</b> NON-MEDICAL
<h2>FOLDING PARTICULATE PROTECTIVE MASK</h2>		
✓ Skin Comfort    ✓ Stronger Protection    ✓ Stereo Shaping		
<h3>INDICATIONS FOR USE</h3> <p>The protective mask is intended used for protection against solid and oily particulates, liquid and microorganisms such as coal dust, cement dust, paint fog, oil mist. Single-use.</p>		△ After use, please discard the mask following local regulation. △ The mask is only for single use, do not use repeatedly.
<h3>LIMITATIONS</h3> <ul style="list-style-type: none"> <li>Not suitable for working places where open flames exist (such as welding, casting, etc).</li> <li>It is not suitable for professional protective use such as oxygen deficient, toxic gas environment.</li> <li>No medical use.</li> </ul>		<h3>STORAGE CONDITION</h3> <p>The product should be stored in a well ventilated, dark and dry environment. Keep away from fire, pollutants and possible pollution sources. also, the transport storage's temperature: -20°C~+40°C, relative humidity: &lt;80%RH.</p>
<h3>CHECKS PRIOR TO USE</h3> <p>Please check whether the packaging is damaged, whether the mask body is damaged, deformed or has other obvious defects before use.</p>		<h3>TRANSPORT CONDITIONS</h3> <p>When transporting this product use original packaging provided. Heavy pressure, direct sunlight, rain and snow should be prevented.</p>
<h3>FITTING</h3>  <ul style="list-style-type: none"> <li>Keep the Nose bridge upward and use your hands extending the straps.</li> <li>Place the respirator on your face to cover the nose, mouth and chin.</li> <li>Attach each straps at the back of the head.</li> <li>According to the user's preference and tightness, they can choose to wear the mask with an antislip buckle to change the straps' length.</li> </ul>		<h3>IMPORTANT NOTICE</h3> <ul style="list-style-type: none"> <li>It is applicable to protect against particles except for escape purpose, and only applicable to personal protection.</li> <li>This filtering half masks is designed to protect against particles, it has passed EN149:2001+A1:2009 to show conformity to the EHSR of Regulation (EU) 2016/425.</li> </ul>
<h3>USE CAUTIONS</h3> <ul style="list-style-type: none"> <li>△ Please check fit of mask prior to use.</li> <li>△ It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.</li> <li>△ Please do not use the mask under low air quality condition, such as serious contaminants in air, oxygen deficiency.</li> <li>△ Do not use the mask in explosive atmosphere.</li> </ul>		<p>FFP2 NR defined in EN 149:2001+A1:2009                  Size: 16.5cm*10.5cm (Dimension tolerance is ±0.5cm)                  Compliant to EN Standards: EN 149:2001+A1:2009                  Production batch No: 2020091601                  Production date: 16/09/2020</p> <p>  <b>Manufacturer:</b> Han Zhaoqing Sporting Goods Company Limited  <b>Address:</b> NO.2-1, Kangtai Street, High-tech zone, Zhaoqing City, Guangdong Province, P.R. China  <b>Website:</b> www.bohansports.cn  <b>Tel:</b> 86-758-3980938             </p>
<p>MADE IN CHINA</p>		<p>See information supplied by the manufacturer</p>
<p>2020091601    Sep 2020    Sep 2022</p>		<p>  </p>
<h1>CE0370</h1>		<h2>1 Pieces</h2>